

# Imagine

whipped cream...



...then realize that it is

## **JET BLUE BITE**

*Bite Registration Material*

- Stays exactly where you want it
- No resistance to closure, no rebound!
- Rock hard for accuracy, yet flexible for easy trimming
- Pick your ideal speed
  - **Fast** - 30 second working time  
40 second oral setting time
  - **Superfast** - 20 second working time  
30 second oral setting time

# Exceptional Handling with firm, accurate, fast set

JET BLUE BITE fast / superfast bite registration material flows with a smooth whipped-cream consistency exactly where you want it in a predictable quantity and ideal ribbon pattern. It offers a fast oral setting time

and sets rock hard for greater accuracy but is still flexible enough for easy trimming or cutting. The superior handling characteristics are the key for a perfect occlusal fit on the first try.

## Direct application of JET BLUE BITE fast



Application of JET BLUE BITE fast



Direct application with the unique Spreader Tip



Ideal working-time



Bite registration



Final bite registration



Easy cutting



Easy trimming



Important and helpful accessories: Unique spreader tip for predictable and consistent dispensing ribbon width and depth

JET BLUE BITE	Working time	Oral setting time	Endhardness	Dimensional stability in %
JET BLUE BITE fast	max. 30 s	min. 40 s	89 Shore A	-0,08%
JET BLUE BITE superfast	max. 20 s	min. 30 s	89 Shore A	-0,12%

Internal measurement

JET BLUE BITE fast 50 ml Single Pack	6491
Cartridge	1 x 50 ml
Mixing Tip Pink	6 pcs.
Spreader Tips	12 pcs.
JET BLUE BITE fast 50 ml Refill	6495
Cartridges	4 x 50 ml
Mixing Tip Pink	12 pcs.
Spreader Tips	12 pcs.

JET BLUE BITE superfast 50 ml Single Pack	6493
Cartridge	1 x 50 ml
Mixing Tip Pink	6 pcs.
Spreader Tips	12 pcs.
JET BLUE BITE superfast 50 ml Refill	6497
Cartridges	4 x 50 ml
Mixing Tip Pink	12 pcs.
Spreader Tips	12 pcs.

JET BLUE BITE Accessories		
50 ml System		
Mixing Tip Pink	40 pcs.	6551
Spreader Tips	100 pcs.	6223
Dispenser 50/75	1 pc.	6900



© 2012 Coltène/Whaledent AG – www.coltene.com

**Coltène/Whaledent Ltd.**  
The President Suite  
Kendal House, Victoria Way  
Burgess Hill, West Sussex  
RH15 9NF / UK  
Tel +44 1444 235486  
Fax +44 1444 870640  
info.uk@coltene.com

**Coltène/Whaledent AG**  
Feldwiesenstr. 20  
9450 Altstätten/Switzerland  
Tel. + 41 71 757 53 00  
Fax + 41 71 757 53 01  
info.ch@coltene.com

**Coltène/Whaledent Inc.**  
235 Ascot Parkway  
Cuyahoga Falls, Ohio 44223/USA  
Tel. + 1 330 916 8800  
Fax + 1 330 916 7077  
info.us@coltene.com

**COLTENE®**