Stainless steel hand files

Instructions for use

EN

Hand root canal instruments: Medical device class I

List: Reamers, K-Files, H-Files

Indication for use

The hand root canal files are used to clean, to disinfect and to shape the dental root canals.

Warnings

1. Only dentists are allowed to use these instruments for the root canal treatment.
2. Be sure to sterilize products before first use and re-use.

Description

All the root canal instruments manufactured under ISO 3630 specifications are aimed to be used by professionals of dental art only during root canal treatment. Professionals learn to use these instruments and the related medical procedures during their formation.

All instruments are made from high grade raw material and machined to ISO standards 3630.

Product example K-File:

Easy identification of instruments with triple coding according to the ISO norm 3630: color code, symbol and number. They are supplied with silicone stopper on the blade. All hand instruments have safety hole on the handle ("parachute system").

Using advices

1. It is essential to follow the steps of cleaning, disinfection and sterilization before first use or for re-use (see our cleaning, disinfection and sterilisation protocol: www.coltene.com).
2. Only for use by dentists.
3. According to its intended use, choose the most appropriate endodontic instruments (type and ISO number) and use it for shape, enlarge, clean and fill the dental root canal.
4. Prior to use, make sure that it is not damaged: no deformations, no rust, and no scratches.
5. Irrigate and introduce the endodontic instruments applying reciprocating and vertical filling movements, into the root canal.
6. The stiffness of hand steel endodontic files helps to negotiate blockages and calcifications.
7. Use this product carefully to avoid puncturing finger.

Packaging

Quantity per pack: 6 units, identical or different (assortment). Plastic box.

Maintenance

1. It is essential to follow the steps of cleaning, disinfection and sterilization before first use or for re-use (see our cleaning, disinfection and sterilisation protocol: www.coltene.com).
2. Dispose the product if damaged or decreased in efficiency.
3. It is the user's responsibility to sterilize products before first use and re-use.
4. For your safety, we recommend using EPIs (Individual Protective Equipment) - for example: Gloves, eye protection.
5. Only use disinfectants with proven effectiveness (EU labeling, FDA approved, ...).
6. After disinfection, put instruments in sterilization sachets. (Check that sachets comply with EN ISO 11607-1 standard).
7. When several instruments are sterilized in a single autoclave cycle, ensure that the maximum load of the autoclave is not exceeded.
8. Follow strictly the instruction manual of the manufacturer of the medical cleaning product.

Storage

No specific requirements as stainless steel is used, however, it is preferable to store devices in a clean, dry place protected from damp.

Life span

The life span is estimated at 3 years from the date of manufacturing but can vary depending on the use (cutting, conditions), on the frequency of use, and on the quantity of sterilization.

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COLTENE/WHALEDENT GmbH + Co. KG