

ISOLATION AND FIXATION

Dental Dam

Upgrade your isolation



 COLTENE

Upgrade Your Isolation

Complete isolation with dental dam is crucial in Endodontic and Restorative dentistry. It helps produce a dry working field to ensure both reliable and high-quality treatment results, protects patients from aspiration or ingestion of foreign objects, and protects soft tissue from contact with other instruments. Studies show that dental dam is also an effective infection control barrier and can

significantly reduce the load of possible infectious airborne particles and minimize saliva and blood contaminated aerosol or spatter.^{1,2} For decades COLTENE has produced a comprehensive portfolio of high-quality dental dam, clamps, and accessories. Upgrade your Isolation with COLTENE.



COMPREHENSIVE

We offer a comprehensive line of products to help isolate in each clinical situation.



SIMPLE

Dental dam application is actually a simple process. Choose the best possible material to meet your specific treatment needs as we guide you through the selection process.



SOLUTION DRIVEN

Our high quality products are engineered to facilitate isolation. Our products and training materials are the bridge that connect solutions with your treatment requirements.

1) Peng, X., Xu, X., Li, Y. et al. Transmission routes of 2019-nCoV and controls in dental practice. *Int J Oral Sci* 12, 9 (2020). <https://doi.org/10.1038/s41368-020-0075-9>

2) Cochran, M. et.al. The efficacy of the rubber dam as a barrier to the spread of microorganisms during dental treatment, *JADA* 119, 1 (1989)

Benefits



Absolute dryness



Unblocked field
of vision on the
treatment area




Safety and
protection



Selection Guidelines

Dental Dam

Our variety of dam materials ensure that the specific requirements of each clinical situation and treatment procedure are successfully and effectively met. Elasticity, the ability to retract, thickness, color and non-allergenic are relevant features to consider when choosing the most appropriate material.

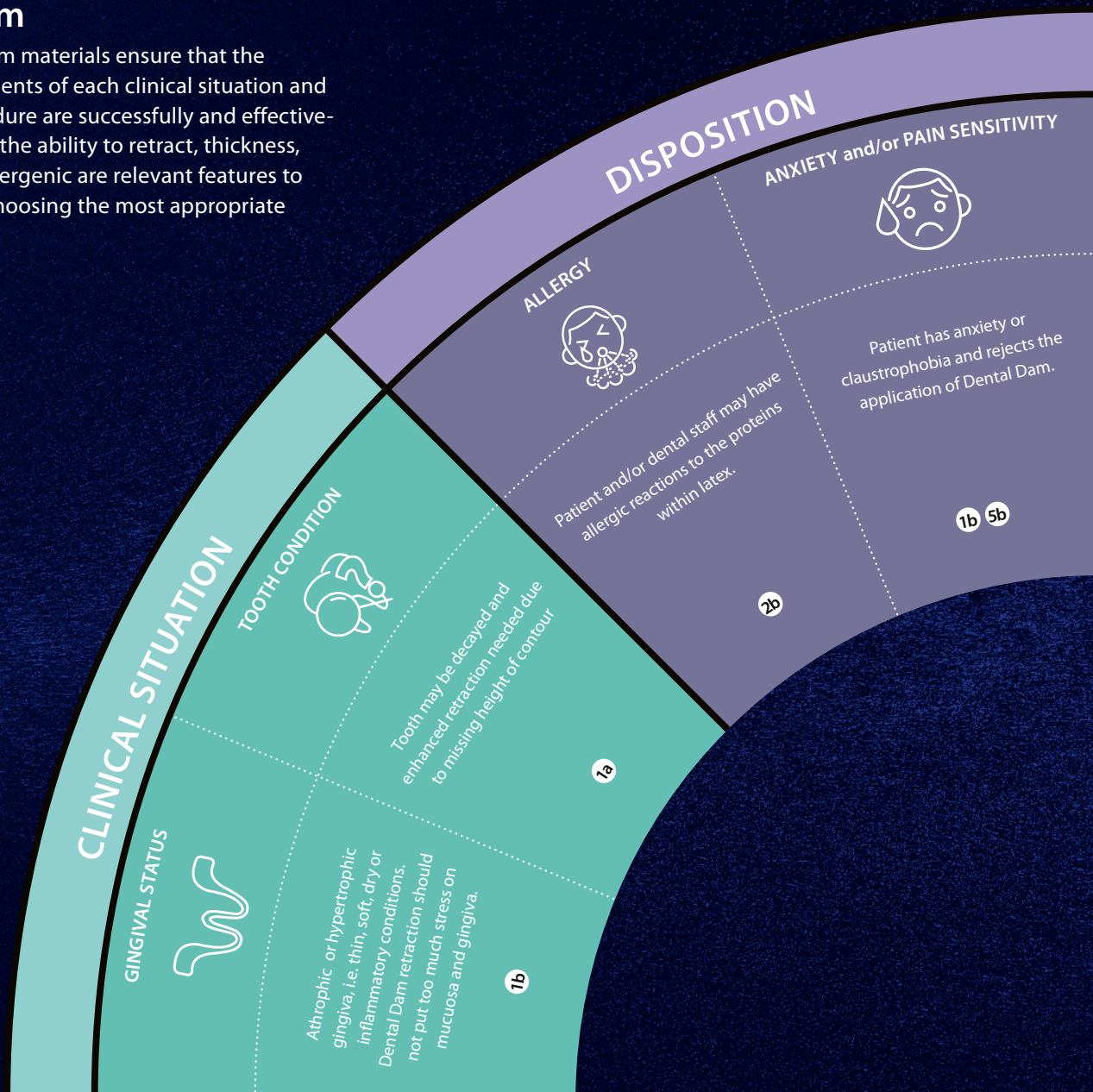

Recommendations

RETRACTION


- 1a**
Choose latex dental dam. Thicker dam allows for better retraction for a tight fit around the isolated area.
- 1b**
Choose a more flexible and thinner dental dam with less retraction which will ease any stress on the gingiva.

ALLERGENS

- 2a**
Choose a powder-free dental dam to reduce the exposure to latex proteins and minimize the risk of developing a latex allergy.
- 2b**
Choose a non-latex dam and avoid allergic reactions on latex proteins.

RESISTANCE

- 3**
Choose latex dental dam as it affords the highest resistance to contact of foreign substances.

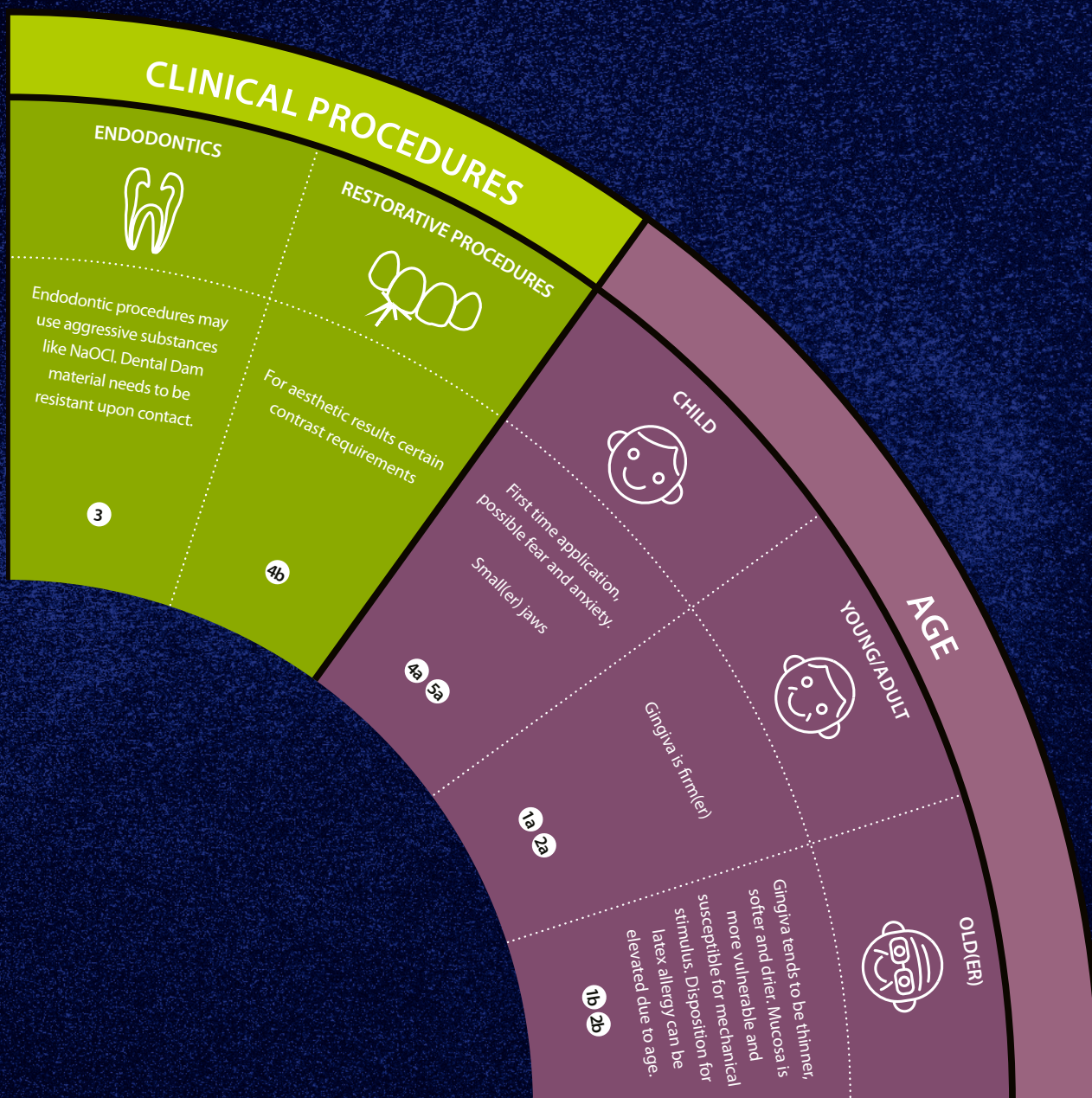

COLOUR OPTIONS

- 4a**
Use psychology and education to take advantage of light and pleasant colours for a calming effect.
- 4b**
Choose a darker dental dam to reduce eye fatigue and provide a good contrast to the working area.

SIZE AND APPLICATION AIDS

- 5a**
Choose 127 x 127 mm dental dam for smaller patients.
- 5b**
Choose Flexi Dam with built-in flexible frame for less stress on the patient.

GENERAL RECOMMENDATIONS

- › Cut out the dental dam sheet so that nose breathing is possible
- › Put Parotisroll on the upper lip to create space between nose and the dental dam sheet
- › Use Dental Dam napkins to ease the contact of the dam material with the skin

Dental Dam Portfolio

Application



NON-LATEX

Dental Dam Non-Latex

Made of a resistant material with latex resembling retraction properties. Ideal for strong retraction.

Flexi Dam Non-Latex

Made of a highly elastic elastomer with reduced tension. It's flexibility facilitates its application on tight contact point. Ideal for soft retraction.

Flexi Dam® Framed Non-Latex


Available with a convenient, built-in frame designed to fit comfortably around the oral cavity while also saving time in the process.



	Dental Dam Non-Latex	Flexi Dam Non-Latex	Flexi Dam Framed Non-Latex
Retraction	×××	×	×
Elasticity	-----	-----	-----
Preframed	-	-	✓
Thickness	medium	medium	medium
Colours	■	■ ■ ■	■
Scent	-	-	-
Children Size 127 × 127 mm	✓	-	-

Stress-strain diagram

Flexi Dam Non-Latex performs with high elasticity while tension is moderate. It is ideal for softer retraction. **Dental Dam Non-Latex** is not as elastic as Flexi Dam while showing higher tension. Retraction forces on the gingiva are higher. It is the latexfree alternative to latex when seeking strong retraction. **Dental Dam Latex** provides best possible retraction

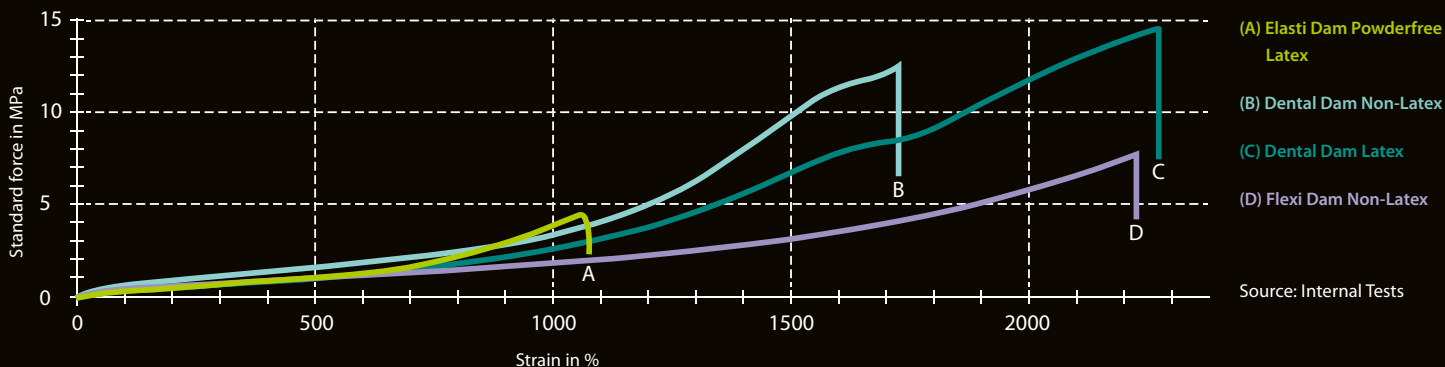

LATEX

Dental Dam Latex

Dental Dam Latex is made of pure, natural rubber latex. Tough and hard to tear, the powdered material is used as a barrier during operative and endodontic procedures.

Fiesta® Dental Dam Latex

Fruit scented latex dam for greater patient and staff comfort. Each box contains an assortment of purple, blue and pink colored sheets.

Elasti Dam® Powderfree Latex

Elasti Dam is an extra elastic and tear resistant latex dam facilitating the application. It is powderfree and has a controlled low level of protein. Patient and dental team are less exposed to allergenic particles and face a lower risk of developing a latex allergy.




Dental Dam Latex	Fiesta Dental Dam Latex	Elasti Dam Powderfree Latex
××××	××××	××
-	-	-
thin, medium, heavy, extra heavy	thin, medium, heavy, extra heavy	medium, heavy
■ ■ ■ ■	■ ■ ■	■ ■
-	✓	-
✓	✓	✓

Order Information


Dental Dam Non-Latex

BRAND	SIZE	COLOUR	THICKNESS	CONTENT
			medium	Sheets
HYGENIC	152 × 152	Green	H09105	15
			H09106	75
HYGENIC	127 × 127	Green	H09928	15

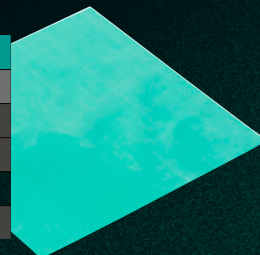

Flexi Dam Non-Latex

BRAND	SIZE	COLOUR	THICKNESS	CONTENT
			medium	Sheets
HYGENIC	152 × 152	Purple	H09945	30
ROEKO			390035	
HYGENIC		Green	H09946	
ROEKO		Blue	390033	

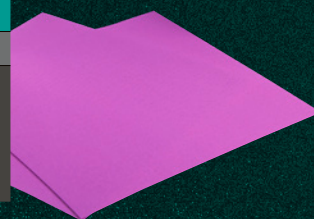



Flexi Dam Framed Non-Latex

BRAND	SIZE	COLOUR	THICKNESS	CONTENT
			medium	Sheets
HYGENIC	100 × 105	Purple Framed	H00750	20



Dental Dam Latex

BRAND	SIZE	COLOUR	THICKNESS				CONTENT	
			thin	medium	heavy	x-heavy		
HYGENIC	152 × 152	Green	H02146	H02147	H02148	H02149	36	
ROEKO			-	390020	390021	390022		
HYGENIC			Dark	H00538	H00539	H00540	H00541	36
				-	H04245	-	-	36
		Blue	-	H03530	H07315	-	36	
			H00533	H00534	H00535	H00536	36	
		Light	-	H04243	-	-	36	
			Green	H02141	H02142	H02143	H02144	52
HYGENIC		127 × 127	Blue	-	H04241	-	-	364
				-	H03529	-	-	52
	Light		H00523	H00524	H00525	-	52	
			-	H04237	-	-	364	

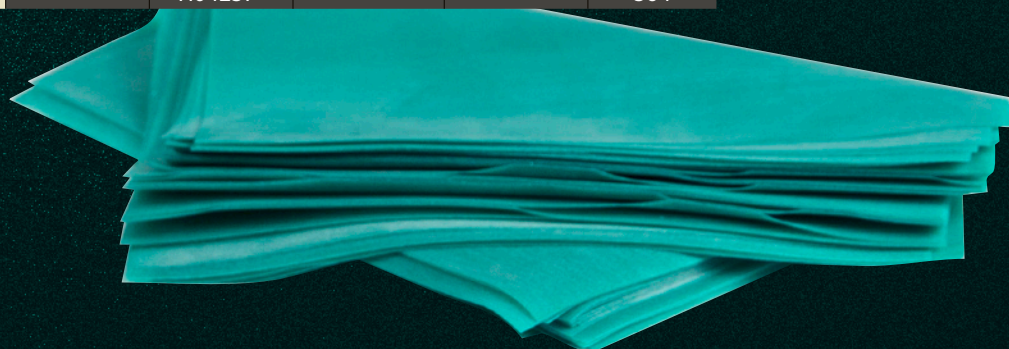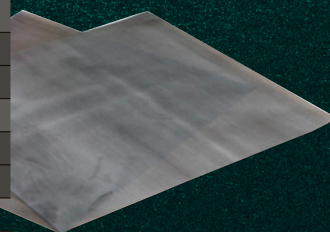


Fiesta Dental Dam Latex

BRAND	SIZE	COLOUR	THICKNESS			CONTENT
			thin	medium	heavy	
HYGENIC	152 x 152	Fiesta	H04639	H04642	H04644	36
			-	H07317	-	360
HYGENIC	127 x 127	Fiesta	H04640	H04641	H04643	52
			-	H07316	-	364

Elasti Dam Powderfree Latex

BRAND	SIZE	COLOUR	THICKNESS		CONTENT	
			medium	heavy		
HYGENIC	152 x 152	Green	H11625	H11676	36	
ROEKO			60013793	60013794		
HYGENIC		127 x 127	Blue	H11627	H11678	52
			Green	H11624	H11675	
ROEKO	Blue		H11626	H11677	36	
			60013795	60013796		

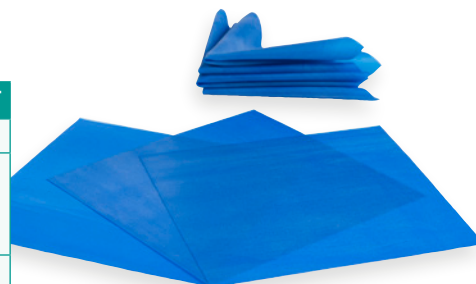

Dental Dam Kits

REF	DESCRIPTION	DENTAL DAM	PUNCH & FORCAPS
H02778	Winged Complete Kit	Latex	Premium
H02900	Winged Simple Dam Kit	Powderfree Latex	Standard
H02790	Wingless Complete Kit	Latex	Premium
H02779	Winged Standard Kit		None

All kits include: Fiesta 9 Clamp Set, plastic frame and Wedjets.



Punch

REF
H01261


Ainsworth-style stainless steel punch with 5 holes for use with Elasti Dam, Flexi Dam, Dental Dam Latex and Dental Dam Non-Latex. Easy to handle and can be sterilized.



Forceps

REF
H01262

Manufactured from high quality stainless steel to enable secure transfer and fixation of a clamp to a tooth. Engineered for a light-weight, comfortable grip and a matte finish to reduce eye fatigue.

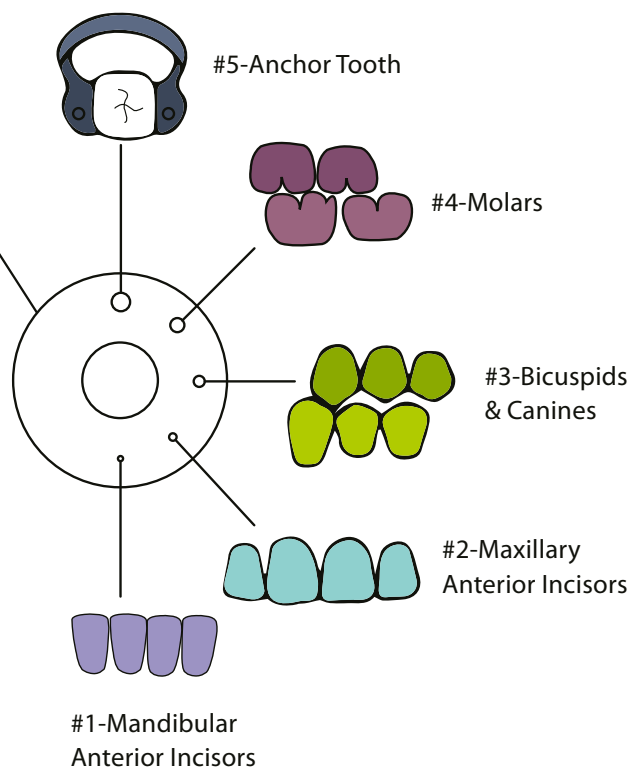


Fixation

Clamps

Dental dam fit partly depends on how it is secured to the teeth. COLTENE makes fixation simple with both Wingless and Winged clamps. Winged clamps have projections at the jaws while wingless do not. Our aim is to accommodate application techniques with any dentist's style. Our dental dam clamps are manufactured from high quality stainless steel for greater durability.


WINGLESS

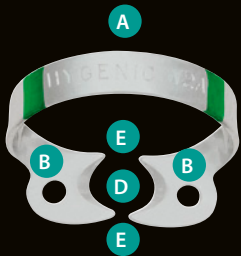

- A Distal Bow
- B Jaws
- C Wings
- D Beak
- E Contact points

Wingless clamps are positioned intra-orally first and then the dam is stretched over the clamp and tooth. Our wingless clamps are designated with a W for easy identification.


WINGED

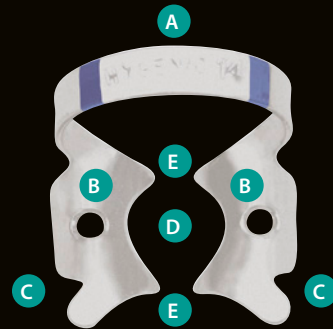

Winged clamps combine steps by placing the dam, clamp and frame simultaneously. Once applied, the dam is stretched off the wings of the clamp.


Four-Point Contact

Select a clamp that will maintain four-point contact with the tooth's proximal surfaces. The clamp is positioned with the bow on the distal of the tooth. Universal clamps can be used on the left and the right side of both the upper and the lower jaw. Both clamp jaws are equal.

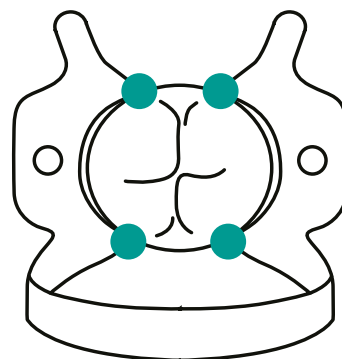

Wedjets – Stabilization Cord 5


Wedjets elastic stabilizing cord provides a quick and easy way to retain dental dam and can complement or supplement clamp usage.

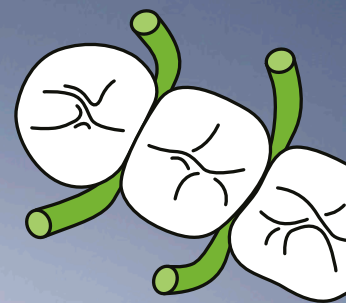

- 1 Standard clamps
- 2 Fiesta colour coded clamps
- 3 Tiger clamps
- 4 Brinker clamps
- 5 Wedjets

Brinker Clamps 4

Provide maximum retraction without laceration of gingival tissue

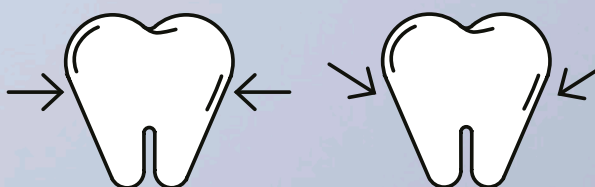

Retention – horizontal effect

Retention – apical effect

Select your clamps

Select a clamp that will maintain four-point contact with the tooth's proximal surfaces. The clamp is positioned with the bow on the distal of the tooth. Universal clamps can be used on the left and the right side of both the upper and the lower jaw. Both clamp jaws are equal. Our comprehensive selection of anatomical shapes help aid in quick and easy placement.

Standard clamps

Gloss finished standard clamps.



#00

Premolar



#00

Premolar

Fiesta clamps

Each clamp is individually color coded for unmistakable identity. Experienced and novice associates will appreciate this time saving identification method when using Fiesta Color Coded Clamps.





#9

Anterior




#7

Molar

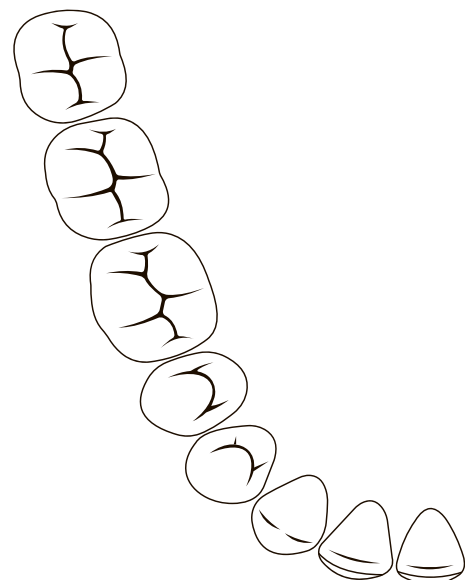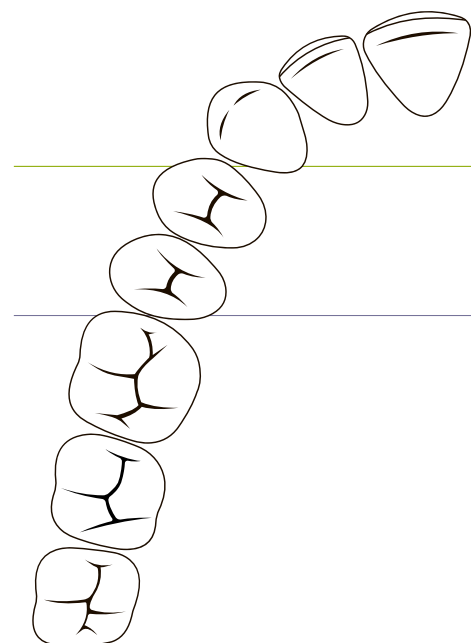

		ANTERIOR SIZES					PREMOLAR SIZES						
Clamps		#9	#212	#B4	#B5	#B6	#00	#1	#2	#2A	#B4	#B5	#B6
Winged	Standard	x					x	x	x	x			
	Fiesta	x							x	x			
	Tiger												
Wingless	Standard	x	x				x		x	x			
	Fiesta	x							x	x			
	Brinker			x	x	x					x	x	x

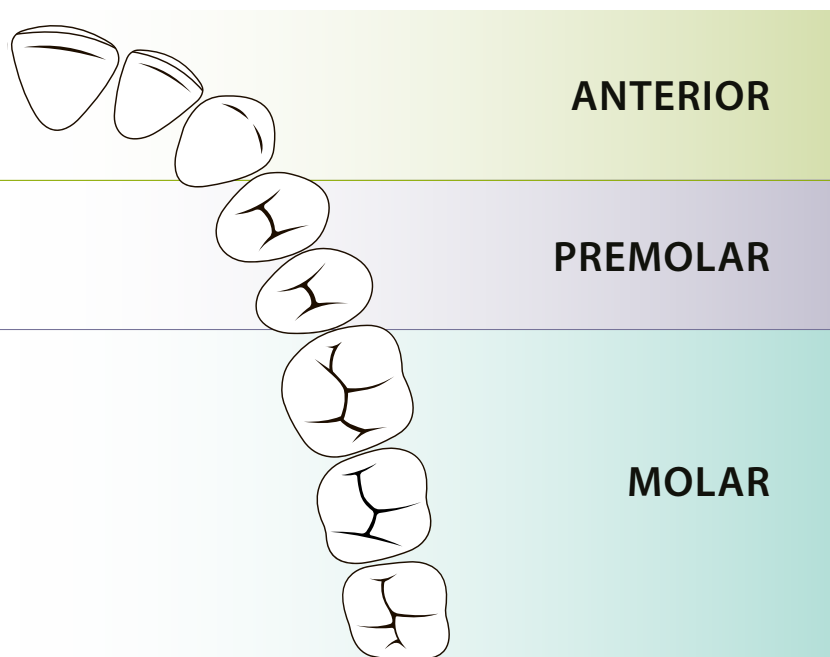

Tiger clamps

Clamps with serrated jaws to be used for structurally compromised tooth.




#12A

Molar
Lower jaw right
Upper jaw left



#13A

Molar
Lower jaw left
Upper jaw right

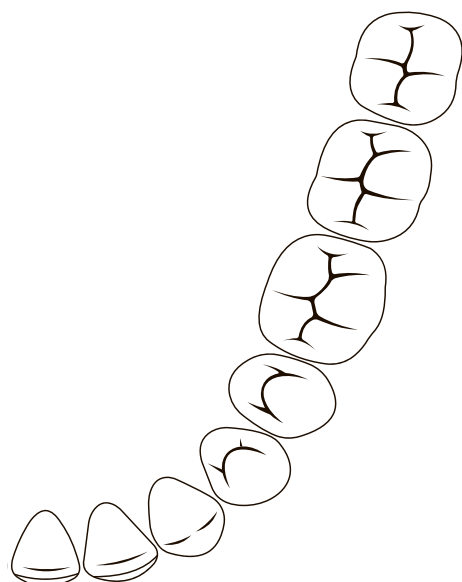

Brinker clamps

Brinker Clamps have extra inclined beaks to provide maximum retraction of the gingiva. They provide maximum retraction without laceration of gingival tissues.



#B1

Molar
Lower jaw right



#B2

Molar
Upper jaw left

MOLAR SIZES													
	#3	#4	#7	#8	#8A	#12A	#13A	#14	#14A	#56	#B1	#B2	#B3
	x	x	x	x	x			x	x				
		x	x		x	x	x		x				
						x	x						
	x		x	x	x			x	x	x			
	x		x	x	x				x	x			
											x	x	x

Order Information

WINGLESS

ANTERIOR



Standard | #9
H02770



Standard | #212
H07792



Fiesta | #9
H02800



Brinker | #B4
H01051



Brinker | #B5
H01052



Brinker | #B6
H01054

PREMOLAR



Standard | #00
H02769



Standard | #2
H02768



Standard | #2A
H02775



Fiesta | #2
H02794



Fiesta | #2A
H02795



Brinker | #B4
H01051



Brinker | #B5
H01052

MOLAR



Standard | #3
H03814



Standard | #7
H02765



Standard | #8
H02767



Standard | #8A
H02766



Standard | #14
H02774



Standard | #14A
H02764



Standard | #56
H03815



Fiesta | #3
H02796



Fiesta | #7
H02797



Fiesta | #8
H02798



Fiesta | #8A
H02799



Fiesta | #14A
H02802



Fiesta | #56
H02803



Brinker | #B1
H01048



Brinker | #B2
H01049



Brinker | #B3
H01050

CLAMP SETS

- H02801 HYGENIC FIESTA 9W Color Coded Wingless Clamp Pak
- H09966 HYGENIC FIESTA 9 Color Coded Winged Clamp Pak
- H09956 HYGENIC FIESTA Color Coded Clamp Organizer
- H02701 HYGENIC Gloss Finish System-9 Winged Clamp Pak
- H01054 HYGENIC Brinker Clamp Set

WINGED



Standard | #9
H02763



Fiesta | #9
H09962



Standard | #00
H02762



Standard | #1
H02703



Standard | #2
H02761



Standard | #2A
H05688



Fiesta | #2
H09957



Fiesta | #2A
H09958



Standard | #3
H05689



Standard | #4
H02704



Standard | #7
H02758



Standard | #8
H02760



Standard | #8A
H02759



Standard | #14
H05690



Standard | #14A
H02757



Tiger | #12A
H02706



Tiger | #13A
H02705



Fiesta | #4
H09959



Fiesta | #7
H09960



Fiesta | #8A
H09961



Fiesta | #12A
H09963



Fiesta | #13A
H09964



Fiesta | #14A
H09965

Accessories

Order Information

Wedjets – Stabilization Cord


SIZE	LENGTH	COLOUR	REF
LATEX			
x-small	2.1 m	blue	H08040
small		yellow	H06522
large		orange	H06523
NON-LATEX			
small	2.1 m	lime green	H06524
large		aqua blue	H06525



Frames

TYPE	SIZE		REF
Ostby plastic frame – radiolucent	6"	152 mm	H00560
U-shaped plastic frame – radiolucent			H01414
U-shaped metal frame			H07083
U-shaped plastic frame – radiolucent	5"	127 mm	H01416
U-shaped metal frame			H07082

All frames may be autoclaved

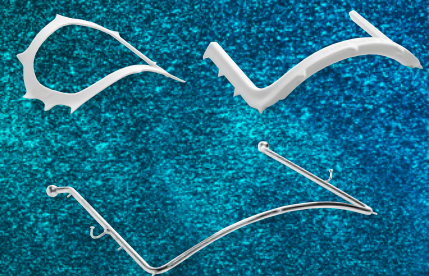

Template

TYPE	REF
6 x 6 Dam Template	H01441
5 x 5 Dam Template	H01440





Ora Shield

TYPE	CONTENT	REF
Regular – for frame sizes 5" (127 mm) and 6" (152 mm)	50 pcs	H01415
Large for use with strap holder	50 pcs	H00841




© COLTENE – www.coltene.com

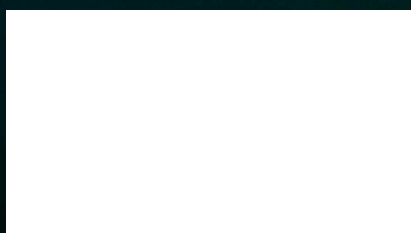

Coltene/Whaledent Inc. 
 235 Ascot Parkway
 Cuyahoga Falls, Ohio 44223 / USA
 T +1 330 916 8800
 F +1 330 916 7077
 info.us@coltene.com

Coltene/Whaledent AG
 Feldwiesenstrasse 20
 9450 Altstätten / Switzerland
 T +41 71 757 5300
 F +41 71 757 5301
 info.ch@coltene.com

Coltene/Whaledent GmbH + Co. KG 
 Raiffeisenstraße 30
 89129 Langenau / Germany
 T +49 7345 805 0
 F +49 7345 805 201
 info.de@coltene.com

