



✗ Rewound (discard)



✓ After use



✓ After autoclaving



✓ New file

## HyFlex<sup>TM</sup> CM

CROWN-DOWN TECHNIQUE - STEP BY STEP CARD

**Coltène/Whaledent GmbH + Co. KG** 

89129 Langenau/Germany

**Coltène/Whaledent Inc.**

Cuyahoga Falls, OH 44223/USA

[www.coltene.com](http://www.coltene.com)

Wiederaufbereitung/Reprocessing procedures/




[www.hyflex.info](http://www.hyflex.info)

CROWN-DOWN TECHNIQUE  
STEP BY STEP CARD


Examples for selection of file sequence depending  
on the clinical situation

SMALL/MEDIUM  
CANALS

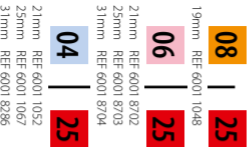

04/25 GuttaPercha Point  
REF 361 725

MEDIUM/LARGE  
CANALS

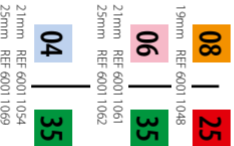

04/35 GuttaPercha Point  
REF 361 735

← Orifice Opener

Varying the taper  
using a constant size

Take each file down  
to resistance

Use a slow speed hand-  
piece at 500 rpm.  
Recommended torque  
setting 2.4 N-cm / 24mNm